

EASYPPOINT®

retractable needle

- AUTOMATED NEEDLE RETRACTION
- EASY TO USE - REQUIRES MINIMAL TRAINING
- MAY BE ACTIVATED BEFORE OR AFTER REMOVAL FROM PATIENT
- PRE-REMOVAL ACTIVATION REDUCES THE RISK OF EXPOSURE TO THE CONTAMINATED SHARP
- ACTIVATION CAN BE ACHIEVED IN A VARIETY OF WAYS
- EFFECTIVELY REDUCES THE RISK OF NEEDLESTICK INJURIES
- PREVENTS RECAPPING AFTER USE
- REMAINS SAFE THROUGH DISPOSAL

compatible with both luer lock & luer slip syringes



may be used with pre-filled syringes



EASYPPOINT®

RETRACTABLE NEEDLES

(50 per box/400 per case)

Now available in
NEW sizes

Catalog # Needle Size

82091	25G x 5/8"
82011	25G x 1"
85011	25G x 1 1/2"
82031	23G x 1"
85031	22G x 1 1/2"
85051	21G x 1 1/2"
85081	18G x 1"
85091	18G x 1 1/2"

OTHER AVAILABLE SAFETY PRODUCTS

VanishPoint® Syringe

VanishPoint® Blood Collection Tube Holder

VanishPoint® Blood Collection Set

VanishPoint® IV Catheter

Patient Safe® Syringe



511 Lobo Lane, Little Elm, TX 75068-5295

P 972 • 294 • 1770 TF 1 • 888 • 703 • 1010 F 972 • 294 • 4400
rtiservice@vanishpoint.com



©2015, 2019 Copyright Retractable Technologies, Inc. All rights reserved.

M098 REV.5 EN, AUG 2019

EASYPPOINT®

retractable needle



— **EASY TO USE** —


- minimal change in injection technique
- allows for single-handed activation
- **clinician's hands and fingers remain behind the exposed sharp** at all times




1
Attach EasyPoint needle to any luer lock or luer slip syringe. Administer medication according to institutional policy.



2
Push the color-coded tab, allowing the needle to automatically retract into the safety chamber. **This device is designed for activation before or after removing the needle from the patient.**

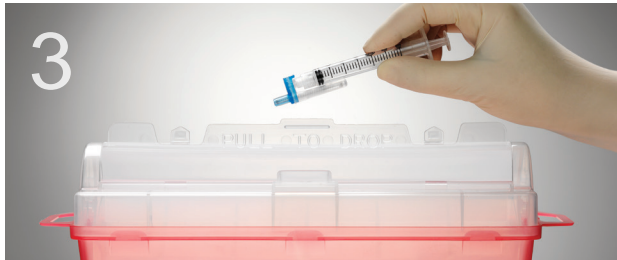

3
Dispose in an appropriate sharps container per protocol of institution.

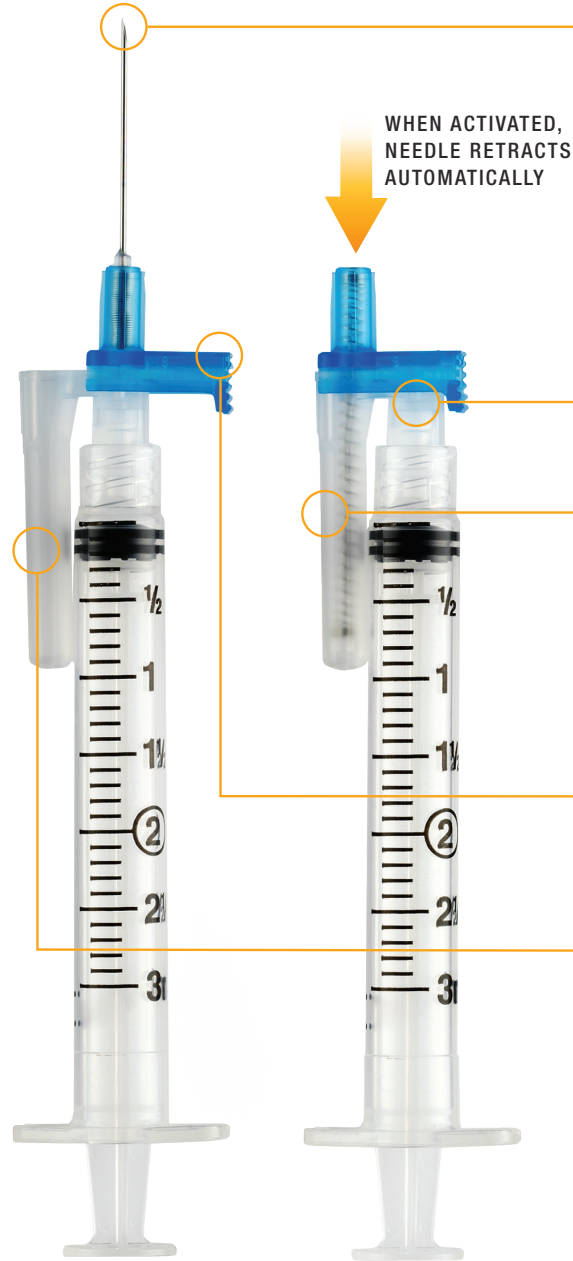

Triple beveled, lubricated needle provides patient comfort

WHEN ACTIVATED, NEEDLE RETRACTS AUTOMATICALLY

Standard luer fitting provides secure, leak-resistant connection

After retraction, contaminated needle is enclosed in safety chamber preventing exposure

Needle hub & activation tab are color-coded for needle gauge

Clear, compact safety chamber does not interfere with injection

AUTOMATED RETRACTION
— now with —
ADDED VERSATILITY

EASYPOINT[®]