



## Needlestick prevention that **meets your needs**

- ▶ Virtually eliminates exposure
- ▶ Single-handed, passive activation
- ▶ Hands remain behind the sharp
- ▶ Integrated safety mechanism
- ▶ Prevents recapping and needle removal
- ▶ Remains safe through disposal
- ▶ Easy to use, requires minimal training
- ▶ Safe and effective for patients

### Catalog No. Description

<b>VanishPoint® U-100 insulin syringes</b> 100 per box/800 per case	
15391 0.5mL VanishPoint insulin syringe (half-unit)	30G x 3/16" <b>NEW</b>
15371 0.5mL VanishPoint insulin syringe (half-unit)	30G x 5/16" <b>NEW</b>
15291 0.5mL VanishPoint insulin syringe	30G x 3/16"
15271 0.5mL VanishPoint insulin syringe	30G x 5/16"
15221 0.5mL VanishPoint insulin syringe	30G x 1/2"
10291 1mL VanishPoint insulin syringe	30G x 3/16"
10271 1mL VanishPoint insulin syringe	30G x 5/16"
10281 1mL VanishPoint insulin syringe	29G x 5/16"
10211 1mL VanishPoint insulin syringe	29G x 1/2"

### **1mL VanishPoint® syringes** 100 per box/800 per case

10131 1mL VanishPoint tuberculin syringe	27G x 1/2"
10151 1mL VanishPoint tuberculin syringe	25G x 5/8"
10161 1mL VanishPoint syringe	25G x 1"

### **VanishPoint® Allergy Syringe Tray** 25 per tray/1000 per case

10134 1mL VanishPoint allergy syringes	27G x 1/2"
--	------------

### **3mL VanishPoint® syringes** 100 per box/600 per case

13021 3mL VanishPoint syringe	27G x 1 1/2"
10301 3mL VanishPoint syringe	25G x 5/8"
10391 3mL VanishPoint syringe	25G x 1"
13011 3mL VanishPoint syringe	25G x 1 1/2"
10311 3mL VanishPoint syringe	23G x 1"
10321 3mL VanishPoint syringe	23G x 1 1/2"
10331 3mL VanishPoint syringe	22G x 1"
10341 3mL VanishPoint syringe	22G x 1 1/2"
10351 3mL VanishPoint syringe	21G x 1"
10361 3mL VanishPoint syringe	21G x 1 1/2"
10371 3mL VanishPoint syringe	20G x 1"
10381 3mL VanishPoint syringe	20G x 1 1/2"

### **5mL VanishPoint® syringes** 100 per box/600 per case

<b>10mL VanishPoint® syringes</b> 100 per box/600 per case	
--	--

**5mL and 10mL VanishPoint® syringes are available with a variety of needle lengths and gauges. For a complete list, please contact RTI or visit our website at [www.retractable.com](http://www.retractable.com).**



**RETRACTABLE TECHNOLOGIES, INC.**  
www.retractable.com

CE  
2797

© 2008, 2019 Retractable Technologies, Inc.  
All rights reserved. M005 REV/15 EN, JUL 2019

511 Lobo Lane  
Little Elm, Texas 75068-5295 USA  
Tel.: (972) 294-1770  
Fax: (972) 294-4400  
Toll Free: 1-888-703-1010  
rtisales@vanishpoint.com



# VANISHPOINT®

SYRINGES

**The new standard for safety®**



Effectively reduces the risk of needlestick injuries

# THE VANISHPOINT® ADVANTAGE

Effective safety with automated retraction technology




1 Use standard procedure for drawing up medication.



2 Prepare and give injection using aseptic technique according to institutional policy.




3 While the needle is still in the patient, fully depress the plunger to activate retraction.

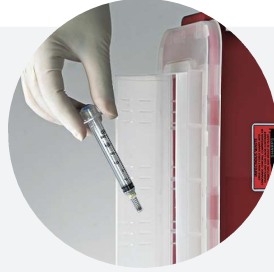

4 Dispose of VanishPoint® syringe in an appropriate sharps container.


Clear, unobstructed calibrations allow for accurate dosing

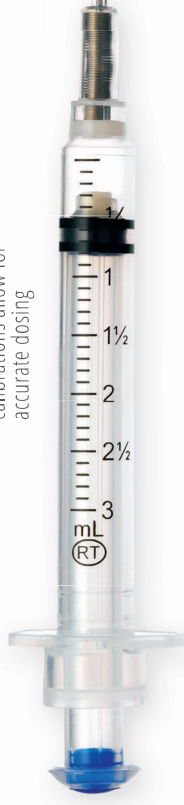

**EASY, ONE-HANDED ACTIVATION**

Triple beveled, lubricated needle provides patient comfort

**NO CONTAMINATED SHARP EXPOSED**

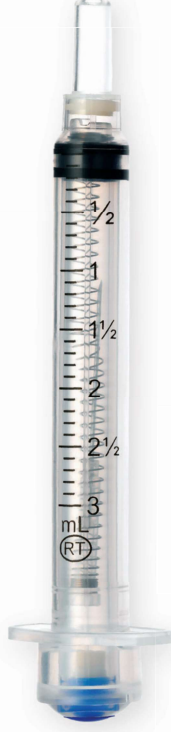


Needle retracts directly from patient into syringe

Pre-removal activation prevents exposure to contaminated sharp



VanishPoint® insulin syringes  
**NOW AVAILABLE WITH**

- 5mm and 8mm needles
- Half-unit increments for accurate small dose delivery

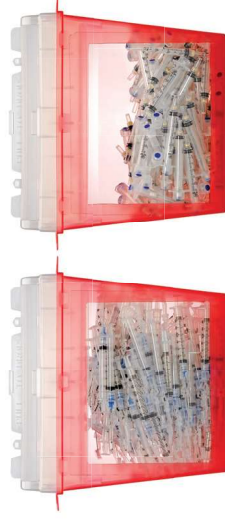

Activated VanishPoint® syringes require less disposal space than other safety syringes and prevent disposal-related injuries.